**Milex® Incontinence Dish with Support  
Fitting Kit Replacement Pessary**Fitting Instructions for the  
Health Care Professional**FITTING  
PESSARIES  
ARE NOT  
TO BE  
WORN MORE  
THAN  
15 MINUTES!****CAUTION:** U.S. Federal law restricts this device to sale  
by or on the order of a physician.**CooperSurgical**

Made in the USA

95 Corporate Drive  
Trumbull, CT 06611 USA  
Phone: (800) 243-2974  
Fax: (800) 262-0105International  
Phone: (203) 601-9818  
Fax: (203) 601-4747**EMERGO EUROPE**  
Molenstraat 15  
2513 BH, The Hague  
The NetherlandsCooperSurgical, Inc.  
95 Corporate Drive  
Trumbull, CT 06611 USA

www.coopersurgical.com

MXFITDISH-DFU • Rev. A • 6/13

**GENERAL INFORMATION****PURPOSE**

The Milex® Pessary Fitting Kit was developed to help you as a health care professional determine accurate pessary type and size prior to prescribing a pessary for your patient.

**CONTENTS**

The pessaries included in this kit are: Incontinence Dish with Support, Ring with Support, Donut, Gellhorn and Cube. The shaded area indicates sizes recommended for pessary inventory, as they are the most common sizes. With these sizes you will be able to fit approximately 85% of patients. Also included in this kit are a foam model and carrying case.

**Warning: Fitting pessaries are not to be worn more than 15 minutes!****LATEX-FREE CURRENT SILICONE INFORMATION:**

CooperSurgical has had nontoxic silicone gynecologic devices available for hyper-allergic patients for more than 10 years.

**ADVANTAGES OF SILICONE:**

1. Longer shelf life and use life
2. Can be autoclaved

**PESSARY CARE:**

Pessaries are powdered with food-grade powder. The powder must be washed off with a mild soap and the pessary thoroughly rinsed with water and dried prior to initial use.

Prior to insertion of any pessary, it is important to thoroughly clean and remove any matter that may be on the outer surface. CooperSurgical recommends washing with mild soap and thoroughly rinsing with water prior to initial use.

The yellow Milex fitting pessaries are designed to be used in the physician's office for selecting and fitting the correct pessary for individual patient use and should only be worn in 15-minute increments, as many of the fitting pessaries do not have nylon pegs for support. The yellow fitting pessaries should be cleaned and sterilized prior to fitting each patient but do not need to be stored in a sterile environment.

With the realization that many different techniques are practiced in the health care community, we attempted to optimize methods to afford the best coverage for our customers. Every facility should validate its own equipment and parameters before processing any medical device.

**RECOMMENDED CLEANING BEFORE INSERTION**

- Decontamination/Disinfection: CIDEX® OPA for 12 minutes
- The agent **must** be thoroughly rinsed off with water.

**RECOMMENDED STERILIZATION BEFORE INSERTION**

Pre-vacuum: 270 °F - 275 °F (132 °C - 135 °C) for 4 minutes

Gravity displacement: 250 °F - 255 °F (121 °C - 124 °C) for 40 minutes

**FITTING INSTRUCTIONS FOR THE HEALTH CARE PROFESSIONAL****IMPORTANT**

- Pessaries are fitted by trial and error. There are no mechanical devices available that can accurately determine the size or type of pessary your patient requires to obtain the desired results.
- Fitting diaphragms **should not** be used to measure the size pessary a patient will need. Diaphragms fit differently from pessaries.
- Even before fitting a pessary, the patient should be informed that it is not uncommon to have to change the size or type of pessary more than once after being originally fitted. This is why it is so important that your patient be instructed to return within 24 hours of the initial fitting and again in 72 hours. Thereafter, re-examination every few months is recommended to ensure that a proper fit is maintained as long as the patient is wearing the pessary.
- At each visit the pessary should be removed and the vaginal vault inspected for signs of allergic reaction or undue pressure.
- At the physician's discretion, the patient can be instructed in the proper removal, cleaning and reinsertion techniques for her own pessary. This process can be performed nightly by the patient under ideal circumstances.
- The INFLATOBALL™, CUBE and TANDEM-CUBE pessaries are the three exceptions to the above. Patients must be instructed to insert, remove and clean these pessaries daily.
- The CUBE and TANDEM-CUBE pessaries fill the entire vaginal vault and have no area for drainage. The INFLATOBALL pessary should not be left in place more than 24 consecutive hours.
- A noncompliant patient should not be fitted with any pessary. It is essential that your patient understands the importance of these frequent follow-up visits and that she fully cooperates with you to ensure the desired results

**Note:** Each type of pessary is available in a wide range of sizes – most pessaries are made in 7 to 14 different sizes.

Experience has shown that in order to properly fit a pessary, you should have at least one of the most commonly used sizes of the pessary (see the size charts).

**Note:** When you purchase the pessaries for your patients, detailed fitting instructions are included in the box for your use along with pessary instructions for the patients.

**HAVE PATIENT REPORT ANY OF THESE SYMPTOMS:**

- Any changes in the color or consistency of vaginal discharge
- Any increase in amount of vaginal discharge
- Any foul odor associated with vaginal discharge
- Vaginal itching

**Complete Prolapse.** Pessaries recommended for third-degree prolapse/proctientia are: Inflated Silicone Donut, 95% Rigid Silicone Gellhorn, Flexible Silicone Gellhorn (Folding), Inflatoball, Silicone Cube or Silicone Tandem-Cube.

**Incontinence.** Pessaries recommended for incontinence are: Ring with knob, Incontinence Ring, Ring with Support and Knob, Hodge with Support and knob, Hodge with knob, Hodge with Support, Hodge, Incontinence Dish, Risser, Incontinence Dish with Support, Smith and Gehrung with Knob.

For medical emergencies and for all other medically-related advice, consult your health care professional.

Review separate instructions with patient to establish use regimen.

**Incontinence Dish with Support Fitting Pessary**  
**65 mm, 70 mm, 75 mm, 80 mm**  
**(Sizes 2, 3, 4, 5)**For stress urinary incontinence with mild  
prolapse and mild cystocele

## DESCRIPTION:

The INCONTINENCE DISH WITH SUPPORT silicone flexible fitting pessary is designed for stress urinary incontinence with mild prolapse and mild cystocele. Using this pessary allows a non-surgical approach to stress urinary incontinence. To properly fit a patient with an INCONTINENCE DISH WITH SUPPORT pessary, it is necessary to have available a minimum of four sizes. The shaded area indicates sizes recommended for office fitting (most common sizes). With these sizes you will be able to fit approximately 85% of patients.

CPT CODE

Procedure: 57160

HCPCS: A4562

### Available in the Following Sizes

Diameter
55
60
65
70
75
80
85

Remember for sizing: The incontinence knob available for these pessaries adds 1/2 inch (13 mm) to the diameter.

**Note:** We recommend the INCONTINENCE DISH WITH SUPPORT pessary be inserted and removed by a physician or other health care professional.

Other styles/models of pessaries (all single-patient use only) are available. Consult the CooperSurgical website at [www.coopersurgical.com](http://www.coopersurgical.com) for details.

### Available models not included in fitting kit:

Incontinence Dish

For stress urinary incontinence with mild prolapse

Incontinence Ring

For stress urinary incontinence

## INDICATIONS:

Stress urinary incontinence

- The INCONTINENCE DISH WITH SUPPORT is indicated for stress urinary incontinence with a mild uterine prolapse and mild cystocele.

## CONTRAINDICATIONS:

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis has been suggested as a possible contraindication to pessary use
- Health care professionals should use their own professional judgment as to the advisability of using this pessary in a pregnant patient

## PRIOR TO FITTING:

**An incontinence pessary is best fitted before patient empties her bladder.**

Ulcerations and erosions frequently occur in cases of complete prolapse due to irritation of the exteriorized cervix.

Whenever possible, reducing the mass and treating the irritation are primary steps before using a pessary.

The only method of determining the proper size INCONTINENCE DISH WITH SUPPORT pessary is trial and error. The unique flexible silicone design facilitates insertion and removal. Silicone does not absorb secretions or emit odors.

## INSTRUCTIONS FOR USE

### STEP-BY-STEP FITTING INSTRUCTIONS FOR THIS PESSARY

It is essential that all incontinence pessaries be precisely fitted. Physicians find it advisable to have available the most commonly used sizes of this pessary. In general, the largest pessary that a patient can comfortably accommodate should be fitted.

Perform a normal pelvic examination before inserting or fitting of any pessary. A first approximation of size can be made by using your fingers to determine the approximate width and length (from the posterior fornix to under the symphysis pubis) of the vaginal vault. This will generally get you within a size or two of the proper pessary.

**Note:** If necessary, irrigate the vagina prior to insertion of the pessary. This will cleanse the vagina of excess discharge and secretions. HCPCS Supply Number A4320 (Irrigation Tray with Syringe, any purpose).

- Wear dry gloves. When necessary, lubricate the entering end of the pessary. Hold the pessary as shown in **Figure 1**. Compress the pessary (bringing sides together) as shown in **Figure 1**.

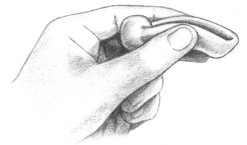
Figure 1

- Use one finger of the opposite hand to depress the perineum. Hold the pessary almost parallel with the introitus (see **Figure 2**). Direct the entering end of the pessary past the cervix into the posterior fornix. Allow the pessary to open into shape after passing the introitus.

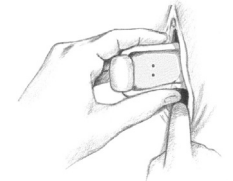
Figure 2

- Use the index finger to guide the pessary along the lower vaginal wall, behind the cervix, and into the posterior fornix.

Figure 3

- Use the index finger to bring the knob up behind the symphysis pubis (see **Figure 3**).

- Have the patient sit, stand and bear down slightly. If there is no leakage of urine and the patient is comfortable with the pessary in position, have her empty her bladder.

- If she is unable to urinate with the pessary in position, remove the pessary and fit her with a smaller size and repeat Step 5.
- If the patient is able to void, continue with Step 6.

- If she can do so without difficulty and the pessary remains in position upon re-examination, and the patient is comfortable with the pessary in place, you have a good indication you have selected the correct size.

- Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.

- The health care professional should be able to sweep one finger between the pessary and vaginal walls. If there is not enough space to do this, the next smaller pessary should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.

- It is sometimes necessary to refit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The use life of a pessary is limited. Examine frequently for signs of deterioration.

**Note:** It is essential that all incontinence pessaries be precisely fitted. Physicians should have available the four most commonly used sizes of this pessary (see the size chart).

*In a limited number of cases, a larger size INCONTINENCE DISH pessary (required to stabilize bladder base) may be difficult to pass through the introitus. In such instances the Millex FOLDING HODGE pessary with knob is an excellent alternative.*

## PATIENT FOLLOW-UP:

Have the patient:

- Report immediately any difficulty in urinating
- Report immediately any discomfort
- Return within 24 hours for first examination
- Return for second examination within 3 days
- Return for examination every 4 to 6 weeks

**Note:** Above schedule of follow-up examinations may be altered to fit the needs of the individual patient.

At each check up, the pessary should be removed and cleaned. If no contraindications are present, the pessary is reinserted.

## TO REMOVE:

Use the index finger to depress the perineum. Hook other index finger under the knob of the INCONTINENCE DISH pessary and pull down. Fold the pessary by bringing the sides together, angling it so that it is almost parallel to the introitus, and gently ease the pessary out.

**Note:** If necessary, irrigate the vagina after removal of the pessary (and before reinsertion) to cleanse the vagina of excess discharge and secretions. HCPCS Supply Number A4320 (Irrigation Tray with Syringe, any purpose).

During each visit, the vagina should be carefully inspected for evidence of pressure or allergic reaction. The patient should be questioned concerning discharge, and disturbance of bowel function or urination. It may be necessary to fit another size or an entirely different type of pessary if these conditions are present.

**DO NOT** assume that a replacement will be the same size as the previous one. Check the fitting to ensure patient comfort and relief of symptoms.

At each checkup, the pessary is removed and cleaned. If there are no contraindications, the pessary is reinserted.

**EXPLANATION OF SYMBOLS**

**REF** Catalog Number

**LOT** Batch Code

**LATEX** Latex Free

**ATTENTION:** See instructions for use.

**Manufacturer**

**Rx ONLY** **CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a physician

**EC REP** Authorized Representative in the European Community.

**CE 0086** Product conforms to the Medical Device Directive 93/42/EEC.


CIDEX® is a registered trademark of Johnson & Johnson.

Millex® and INFLATOBALL™ are trademarks of CooperSurgical, Inc.

CooperSurgical is a registered trademark of CooperSurgical, Inc.

© 2013 CooperSurgical, Inc.

**CooperSurgical's Pessary Product Line**

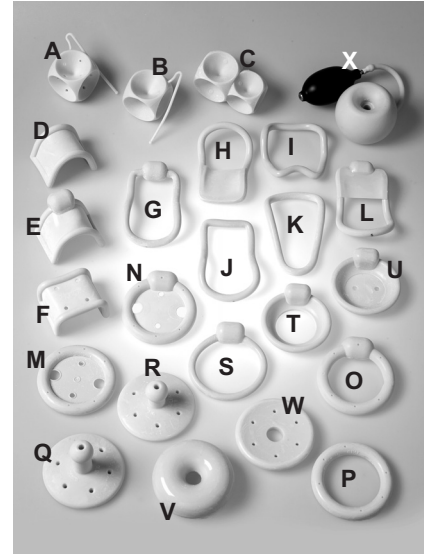- A** Cube w/Drainage Holes
- B** Cube
- C** Tandem-Cube
- D** Gehrung
- E** Gehrung w/Knob
- F** Regula
- G** Hodge w/Knob
- H** Hodge w/Support
- I** Risser
- J** Hodge
- K** Smith
- L** Hodge w/Support & Knob
- M** Ring w/Support
- N** Ring w/Support & Knob
- O** Ring w/Knob
- P** Ring
- Q** Gellhorn - Long Stem
- R** Gellhorn - Short Stem
- S** Incontinence Ring
- T** Incontinence Dish
- U** Incontinence Dish w/Support
- V** Donut
- W** Shaatz
- X** Inflatoball

95 Corporate Drive  
Trumbull, CT 06611 USA  
Phone: (800) 243-2974  
Fax: (800) 262-0105

www.coopersurgical.com

MXFITDISH-DFU • Rev. A • 6/13

**International**  
Phone: (203) 601-9818  
Fax: (203) 601-4747

**EC REP** **EMERGO EUROPE**  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

**CooperSurgical, Inc.**  
95 Corporate Drive  
Trumbull, CT 06611 USA

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International  
Tel.: +1 (203) 601-9818  
Fax: +1 (203) 601-4747

**EMERGO EUROPE**  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

CooperSurgical, Inc.  
95 Corporate Drive  
Trumbull, CT 06611 USA

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