EN

MILEX®
a CooperSurgical Brand

GEHRUNG Folding Pessary

For effective support in cases of cystocele

AVAILABLE MODELS

GEHRUNG **REF** MXPGS (Pessary Only) MKKPGS (Pessary, TRIMO-SAN™, Jel Jector)

GEHRUNG with KNOB **REF** MXPGSK (Pessary Only) MXKPGSK (Pessary, TRIMO-SAN™, Jel Jector)

Single-Patient Use **R_xOnly**

Wash pessary with mild soap and thoroughly rinse prior to initial use.

DESCRIPTION

The Milex® GEHRUNG Pessary is a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.

WARNINGS

- The GEHRUNG pessary with and without the KNOB contain a wire coil which must be removed before X-ray or MRI.
- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use in these pessaries on a patient with a known silicone allergy.
- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material.
- Use only TRIMO-SAN. Other materials have not been tested for compatibility.

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CooperSurgical

INDICATIONS FOR USE

For effective support in cases of cystocele. It can also be manually shaped to reduce a rectocele. The GEHRUNG pessary is more adaptable in cases where a cystocele tends to slip out when other pessaries are used.

- It derives its support from the remnants of the levator sling, laterally. This avoids pressure on the rectum, which, especially in older patients, sometimes causes difficulty with normal bowel action.
- The GEHRUNG pessary also provides broad support under the bladder and prevents the descent of the uterus. It does not interfere with vaginal douching or coitus. The GEHRUNG with KNOB stabilizes the urethrovaginal junction.

CONTRAINDICATIONS

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient

RECOMMENDED CLEANING INSTRUCTIONS*

- Prepare a cleaning solution by mixing a mild soap with tap water using the soap manufacturer’s recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
 - Soak and Scrub
 - Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
 - Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
 - Following scrub, inspect device for visible soil residue.
 - Rinse
 - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
 - Allow the device to dry.

PATIENT INSTRUCTIONS FOR PESSARY USE

CAUTION
COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

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CooperSurgical

- To ensure the desired correction of your condition, your healthcare professional needs your full cooperation.
- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner.
- It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

TRIMO-SAN is available at the prescription counter of your pharmacy. A prescription is required.

REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN:

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge
- Any increase in the amount of vaginal discharge or vaginal bleeding
- Any foul odor associated with vaginal discharge
- Vaginal itching
- If the pessary falls out

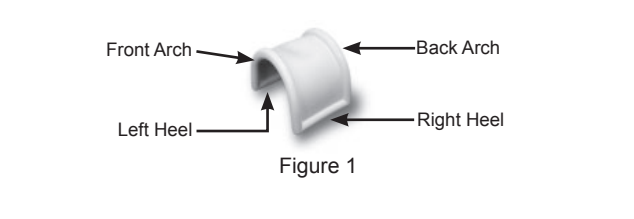
For medical emergencies and for all medically related advice, consult your healthcare professional.

FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

INSTRUCTIONS

- Wear dry gloves. When necessary, the entering end of the pessary can be coated with TRIMO-SAN.
- The pessary is folded and inserted into the vagina with both heels parallel to the pelvic floor, left heel first (see Figure 2).
- Push the pessary past the introitus and then turn the pessary gradually

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Figure 3

Figure 4

by pushing the right heel back and pulling the left heel forward (see Figure 3). Keep both heels parallel to the pelvic floor during the entire fitting.

- Gradually push the back arch over the cervix in the anterior fornix and the front arch behind the symphysis pubis (see Figure 4).
- In fitting the GEHRUNG with KNOB pessary, the knob should rest so that it supports the urethrovaginal junction. If the pessary is comfortable, have the patient empty her bladder. If she is not able to urinate with the pessary in position, fit her with the next smaller size; have her empty her bladder.
- A correctly placed pessary has both heels resting on the pelvic floor with the arches and cross support forming a bridge to raise the bladder as illustrated (see Figure 5).
- Have the patient sit, stand, and bear down. Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.
- NOTE:** If the patient is unable to urinate with the pessary in position, remove it and fit her with the next smaller size. Repeat as necessary.
- If the patient can void without difficulty, pessary remains in position upon re-examination, and the patient is comfortable with the pessary in place, this is a good indication that the correct size may have been selected. Patient experience may vary.
- Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.
- The healthcare professional should be able to sweep one finger between the pessary and the vaginal walls. If there is not enough space to do this, the next smaller size should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.

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CooperSurgical

- It is sometimes necessary to refit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The useful life of a pessary is limited. Examine frequently for signs of deterioration (such as cracks or breaks in silicone outer surface). A pessary should be replaced if damaged.
- Ulcerations and erosions frequently occur in cases of complete prolapse due to irritation of the exteriorized cervix or vaginal wall. Whenever possible, reducing the mass and treating the irritation are primary steps before using a pessary. Prolapse reduction may resolve cervical vaginal irritation. Verification of cervical cytology (pap) and/or biopsy as clinically indicated.
- During each visit, the vagina should be carefully inspected for evidence of pressure or allergic reaction. The patient should be questioned concerning douching, discharge, disturbance of bowel function or urination. It may be necessary to fit another size or an entirely different type of pessary.
- At the physician's discretion, the patient can be instructed in the proper removal, cleaning and reinsertion techniques for her own pessary. This process can be performed nightly or even weekly by the patient under ideal circumstances.

TO REMOVE

- Gradually push back the arch off the cervix while turning the pessary.
- Pull one heel forward while pushing the other heel back.
- Fold the pessary with the arch up, gently pulling the pessary, one heel first through the introitus, keeping both heels parallel to the pelvic floor.

RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

NOTE: The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

Figure 2

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Consult instructions for use

Not made with natural rubber latex

R_xOnly **CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a physician

Manufacturer

CooperSurgical

TRIMO-SAN™ is a trademark of CooperSurgical, Inc. Milex® and CooperSurgical are registered trademarks of CooperSurgical, Inc.

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© 2021 CooperSurgical, Inc.