

# ALLY II UPS® Quick Reference Guide



# ALLY II UPS® Quick Reference Guide


## STEP 1. MOUNTING THE ALLY II UPS

- Place rail clamp on patient's right side near stirrup. Lower mounting tab into rail clamp and tighten.



## STEP 2. READYING THE CONSOLE

- Attach power cord. Connect to wall outlet. Turn on power, verify that green indicator is lit.
- Position patient with buttocks close to edge of table.

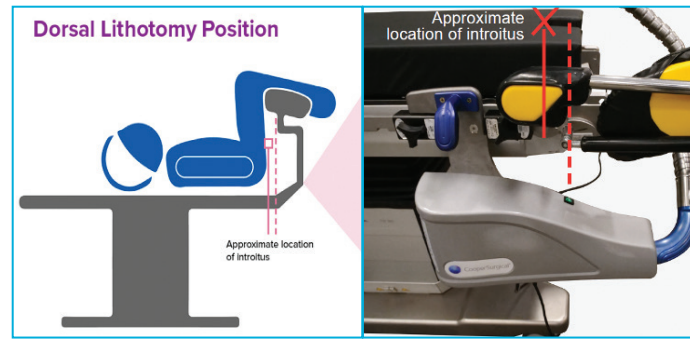

## STEP 3. ATTACHING ADAPTER DRAPE

- Drape ALLY II arm after patient is prepped and before draping patient's legs and under buttocks.
- Choose adapter drape based on UM type.

### RUMI® II System:



### Advincula Arch®:



### Advincula Delineator™:



- (Sterile personnel) Align pinch clips of drape on tip of arm and press to snap in place.



- (Non-sterile personnel) Grasp both drape tabs, pull down to cover arm.




## STEP 4. ATTACHING ALLY II TO MANIPULATOR

- After manipulator is positioned and patient is in Trendelenburg, attach ALLY II arm to manipulator.

- RUMI® II System:**  
Match handle rivet to depression in adapter.



- Advincula Arch®:**  
Align adapter stem with hole in handle.

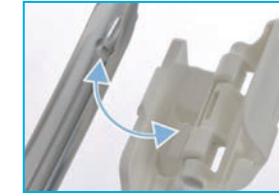

- Advincula Delineator™:**  
Align keel on handle with pocket on adapter.



- Close and lock latch.



## STEP 5. MANIPULATION

- Press foot pedal to release arm for positioning manipulator. Release foot pedal to lock ALLY II in place, holding manipulator and anatomy in a static position.



## STEP 6. REMOVAL

- When procedure is completed, detach manipulator by opening latch.



- Detach adapter drape by squeezing pinch clips.



- Move flexible arm or remove ALLY II from table before raising foot of bed.



- Loosen knob on the rail clamp to remove ALLY II. Lift using blue handle and flexible arm. Place the ALLY II and the rail clamp in the cart for storage.



**Note:** Pressing foot pedal while powering down will keep the arm flaccid during storage. ALLY II may also be stored with flexible arm in a rigid position.



# ALLY II UPS<sup>®</sup> Quick Reference Guide (Cont.)

## CLEANING:

- To avoid damage to the unit, please follow cleaning protocols:
  - Only use Isopropyl Alcohol or CaviWipes™ **not to exceed .28% ammonium chloride.**
  - Ensure flexible arm is locked in upright orientation during cleaning application.
  - Ensure flexible arm is thoroughly dry before storing in flaccid position.



**Important Safety Information:** The ALLY II UPS<sup>®</sup> and the sterile, **Single-Use Adapter Drape** for the ALLY II UPS<sup>®</sup> are intended to secure uterine manipulator devices for uterine positioning and to assist the surgical staff in mounting, positioning, and holding uterine manipulators during gynecological laparoscopic surgical procedures. **CONTRAINDICATIONS:** CooperSurgical manipulator handles should not be used when an IUD is in place, when the patient is pregnant, planned gamete intrafallopian transfer procedures, in patients with suspected pelvic infections or where it is difficult to insert the silicone tip into the cervix or uterus. **WARNINGS:** ALLY II UPS is only intended for use with CooperSurgical's family of uterine manipulators. Read Instructions for Use carefully, as failure to properly follow the instructions could lead to **injury and improper functioning of the device.**  
<https://www.coopersurgical.com/ally-iiifu>

800.243.2974 | 203.601.5200  
[www.coopersurgical.com](http://www.coopersurgical.com)



CooperSurgical<sup>®</sup>