EN
LapSac® Surgical Tissue Pouch

Instructions for Use

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Bruksanvisning

LAPSAC® SURGICAL TISSUE POUCH

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The LapSac Surgical Tissue Pouch is a reinforced pouch with inner liner and drawstring closure. Where applicable, the LapSac Surgical Tissue Pouch may also be available preloaded in an introducer sheath.

INTENDED USE

Used to isolate tissue during, or prior to, surgical removal.

CONTRAINDICATIONS

Not for use with power morcellators.

WARNINGS

- The LapSac Surgical Tissue Pouch is not intended for use with sharp surgical instruments.
- Care should be taken to avoid damaging, puncturing or tearing the LapSac Surgical Tissue Pouch.
- Use caution when removing potentially malignant or harmful tissues, to isolate contamination from surrounding tissue.

PRECAUTIONS

None known

INSTRUCTIONS FOR USE

1. Following aseptic technique, introduce the LapSac Surgical Tissue Pouch into the abdominal cavity.
2. Place tissue to be removed from the body into the pouch.
3. Using the drawstring, carefully close the LapSac Surgical Tissue Pouch. Please note that if excess force is used, the drawstring may break or tear the LapSac.
4. Remove the LapSac Surgical Tissue Pouch from the body, taking care not to spill contents of the bag.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd