
- EN 3 **Charger for Blood Flow Monitor - DP-M350**
Instructions for Use
- CS 7 **Nabíječka pro monitor průtoku krve - DP-M350**
Návod k použití
- DA 11 **Oplader til blodflowmonitor - DP-M350**
Brugsanvisning
- DE 15 **Ladegerät für Blutflussmonitor - DP-M350**
Gebrauchsanweisung
- ES 19 **Cargador del monitor de flujo sanguíneo DP-M350**
Instrucciones de uso
- FR 23 **Chargeur du moniteur de débit sanguin - DP-M350**
Mode d'emploi
- IT 27 **Caricabatteria del monitor per flusso ematico - DP-M350**
Istruzioni per l'uso
- NL 31 **Oplader voor bloedstroommonitor - DP-M350**
Gebruiksaanwijzing
- PL 35 **Ładowarka do monitora przepływu krwi - DP-M350**
Instrukcja użycia
- PT 39 **Carregador para monitor de fluxo sanguíneo – DP-M350**
Instruções de utilização
- RU 43 **Зарядное устройство для расходомера крови — DP-M350**
Инструкция по применению
- SV 47 **Laddare för blodflödesmätare - DP-M350**
Bruksanvisning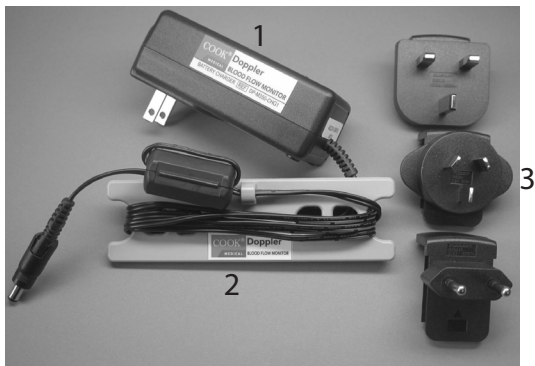

1. Charger
2. Cable Manager
3. Adapter plugs

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Charger for Blood Flow Monitor - DP-M350

Instructions for Use

Read these instructions before using the charger

This product is designed for indoor use only and should not come into contact with water or dust. To prevent overheating the product should not be covered during use.

The mains socket should be easily accessible. In the event of operational error, the plug should be immediately removed from the socket.

The product contains dangerous voltages and the cover should not be removed. All service or maintenance work should be carried out by qualified personnel who can get assistance by contacting the manufacturer's agent.

The unit is double insulated (in insulation class II).

System: The Charger for Blood Flow Monitor DP-M350 (DP-M350-CHG1) is to be used with the Cook Doppler Blood Flow Monitor (DP-M350).

Technical specification: See product labeling.

CHARGING INSTRUCTIONS

1. Do not connect the charger to the mains before it is connected to the monitor.
2. Connect the charger to the mains.
3. When charging is complete, disconnect from the mains before removing monitor connections.

CHARGE INDICATOR (LED)

RED – Indicates battery pack is being charged.

GREEN – Indicates battery pack is fully charged

USE OF ALTERNATE (SUPPLIED) PLUG ADAPTERS

1. The charger (DP-M350-CHG1) for the Doppler Flow Monitor (Model: DP-M350) is supplied with various plug adapters that are configured to connect to regional mains input connections. The charger is designed to operate at input voltages of 100-240VAC 50-60Hz max 0.35A without any action on the part of the user.
2. Prior to using the charger, confirm that the voltage at the mains is compatible with the unit.
3. Select the appropriate plug adapter for the local mains.
4. If the appropriate plug adapter is not connected to the primary charger unit, remove the existing plug adapter by pressing on the grated area directly beneath its output plugs while pushing in the direction of arrow (Figure 1a) or by placing fingers directly below the elevated plug adapter and pushing upward and away from the cable (Figure 1b). The attached plug adapter will detach and slide upward and off the primary charger unit.

5. Orient the appropriate plug adapter so that it is appropriately in-line with the receiving channel of the primary charger unit (Figure 2).
6. Slide the plug adapter fully onto the primary charger unit (Figure 3).
7. Plug into mains as required.

Cable / Cord Manager

The Cable / Cord Manager (as supplied) can be used to organize the charger's cable when the unit is not in use (Figure 4).

Figure 1a

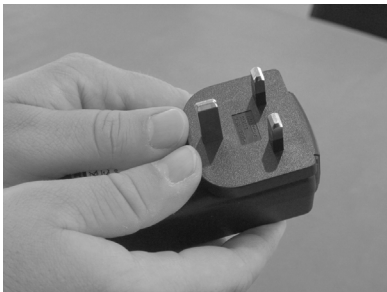
Figure 1b

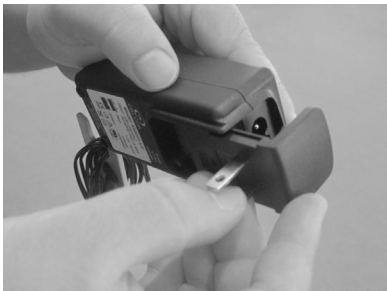
Figure 2

Figure 3

Figure 4

STERILIZATION AND CLEANING

The Charger for the Cook Doppler Blood Flow Monitor may be cleaned with a mild detergent or isopropyl alcohol. Prior to cleaning, make sure that the monitor is not connected to a power supply. Do not immerse the unit in solution or allow solution to enter.

TRANSPORT and STORAGE

Recommended Storage,
Shipping Temperature: -40°C - +60°C

Keep Dry

Recommended Humidity: non-condensing

Keep away
from sunlight

DISPOSAL

To dispose of the charger, please return to Cook or to your Cook Medical products distributor.

As per the WEEE directive, Cook accepts its responsibility to recover and recycle our devices.

- The equipment that you bought has required the extraction and use of natural resources for its production. It may contain hazardous substances that could impact health and the environment.
- In order to avoid the dissemination of those substances into our environment and to diminish the pressure on the natural resources, we encourage you to use the appropriate take-back systems. Those systems will reuse or recycle most of the materials of your end-of-life equipment in a sound way.
- The crossed-out wheeled bin symbol invites you to use those systems.
- If you need more information on the collection, reuse, and recycling systems, please contact your local or regional waste administration.
- You can also contact us for more information on the environmental performances of our products.

For more information on where you can send your waste equipment for recycling, please contact the appropriate European representative below.

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© COOK 2021
VANDERGRIFT, PA 15690 U.S.A.
COOK MEDICAL EUROPE LTD