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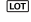
## Symbol Descriptions: English

	In this document, indicates conditions or practices that could lead to illness, injury, or death („WARNING“).
	In this document, indicates conditions or practices that could damage the equipment or other property („CAUTION“).
	Consult Instructions for Use
	Do not re-use, Single use device
	Do not sterilize
	Catalogue Number
	Lot Number
	Use By Date
	Manufacture Date
	Quantity
	Manufacturer's contact information
	Not made with natural rubber latex
	Fragile
	Temperature limit
	Humidity limitation
	For Professional Use
	Do not use if package is damaged

**STERILE** EO Sterilized using ethylene oxide (only applicable to versions marked as sterile on unit packaging)

### Indications for Use

The single-use lighted vaginal speculum is indicated for use on women undergoing a procedure requiring vaginal access and exposure (e.g. for pelvic examination or Pap Smear). The device uses opposing bi-valve plastic blades to separate the vaginal wall and is intended to expose the interior of the vagina and exterior of the cervix.

### Warnings and Cautions

**⚠ WARNING:** Inspect each device for damage prior to use.  
**WARNING:** If a device is damaged, do not use any device from the same box.  
**WARNING:** When in transit or storage, device may be subject to damage beyond the control of the manufacturer or supplier.  
**WARNING:** Never use device with laser equipment.  
**WARNING:** Treat used devices as bio hazardous infectious material. Dispose of used device in suitable disposal unit or in accordance with local regulations.  
**WARNING:** Devices are not compatible with gamma radiation or autoclave sterilization and should never be sterilized by any method.  
**WARNING:** No modification of this equipment is allowed.  
**WARNING – Risk of reuse:** The strength of the device may be significantly compromised and the resulting risk to the patient and/or user is unknown.  
The light may not function properly.  
**CAUTION:** Use by trained personnel only.  
**CAUTION:** U.S. federal law restricts this device to sale by or on the order of a physician.  
**CAUTION:** Light source is single-use and of limited duration. Do not activate light source until you are ready to use the device.

### Warranty

Please visit [www.obpmedical.com](http://www.obpmedical.com) “Terms and Conditions” page for warranty information. OBP Medical should be notified immediately via phone or email of any defective units.

**obp**<sup>TM</sup>  
simple, bright ideas.

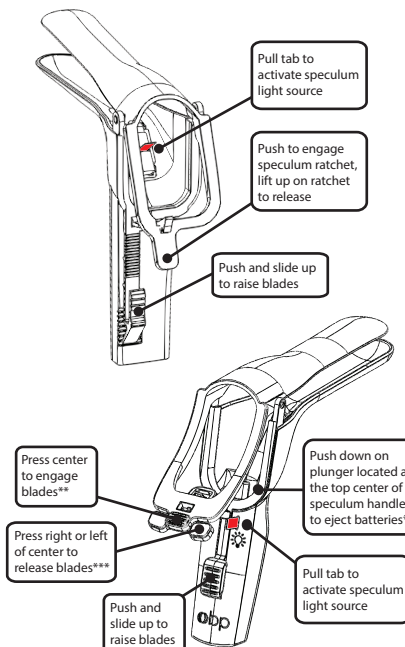
**ER-SPEC**® Single-Use Vaginal Speculum with Integrated LED Light Source

**HER-SPEC**™ Single-Use Click-Free Vaginal Speculum with Integrated LED Light Source

**REF** C020001 thru C029999

Includes **OFFICESPEC**® side-opening model

 **OBP Medical**  
360 Merrimack Street  
Building 9  
Lawrence, MA 01843 USA  
+ 1-978-291-6853  
[www.obpmedical.com](http://www.obpmedical.com)

\*batteries eject downward through handle

\*\*counterforce required to engage blades and lock hinge

\*\*\*counterforce required to close while pressing release button

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